

Remote User Interface

Quick Start Guide

Browser (Laptop)



The versatile screen can display multiple views or just one full screen.



Remote User Interface

Quick Start Guide

Browser (Laptop)

Pan/Zoom Controls

Close: Closes popup to icon

Expand View/Collapse View:

Expands one view to full screen or collapses view to display two views

Zoom Controls:


Zoom views in and out

Pan Controls:

Move views up/down, left/right, or tap middle dot to recenter

Collaboration Tools

Side Chat: Allows Remote Host & Panelist(s) to converse **without** the procedure room hearing the conversation*

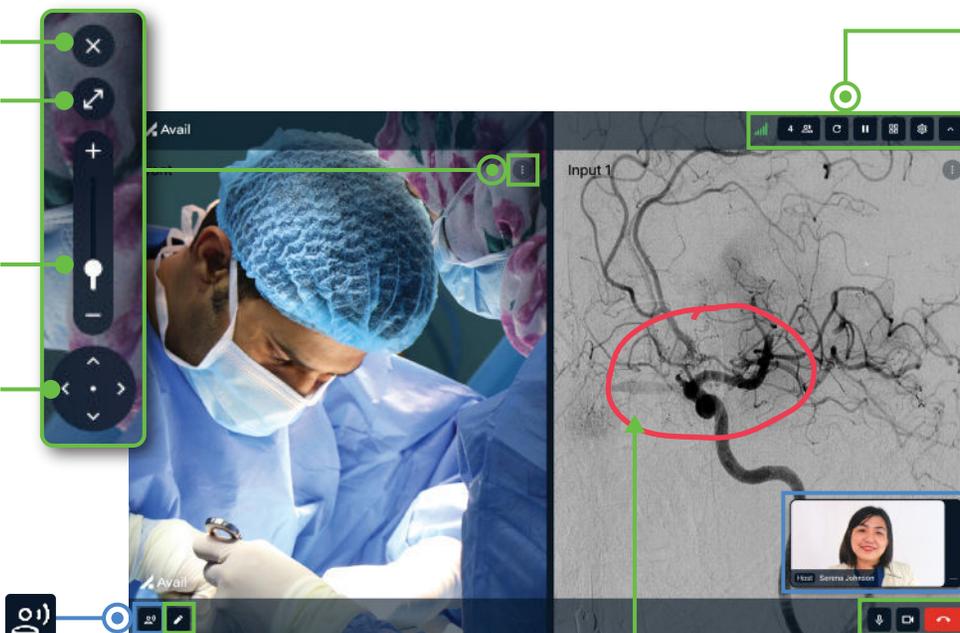

Telestration: Display/collapse Telestration Tools

Width Color Delete

Top Bar Menu

Show/Hide icons: Show/Hide icons

Settings: Access audio settings

Layouts & Presets:

Layouts: Choose one or two views to stream (*inputs, overhead camera, front camera*)

Presets: Save views to return to during a case. Delete preset as needed

Freeze View: Freeze and unfreeze views

Refresh: Refresh network connection

Participants: Number of participants on the Avail event

Network Connection: Network connection strength

Host Video

- **Host** has the ability to display or minimize their video
- Click and drag to **move Host video** around the screen

Call Controls

Mic: Mute/unmute yourself

Video: Start/stop video

Hang Up: Leave event